Distributed by: / Distribuido por:
Lansinoh Laboratories, Inc.
Alexandria, VA 22314, USA +1 800-292-4794

Manufactured by:
Southwest Technologies, Inc.
1746 Levee Road
North Kansas City, MO 64116

2 Gel Pads / 2 Protectores de gel

Open here

Lansinoh

Soothies® Gel Pads

Protectores de gel Soothies®

Caution: Medical device. Use only as directed. Keep out of reach of children. For external use only. Total usage of gel pad therapy should not exceed 3 weeks. If condition persists, discontinue use and seek the advice of a breastfeeding specialist or healthcare provider. Do not use if package is damaged.

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LOT

Directions:

- Open pack & remove gel pad from reusable tray
- Gently place the gel side of the pad directly over nipple to clean dry skin
- Before breastfeeding or pumping, remove gel pad. Inspect breast for residue and gently cleanse if necessary
- After breastfeeding or pumping, reapply gel pad over nipple
- May be used for up to 72 hours or until fully saturated. If pad becomes spongy or less tacky, it is likely saturated and should be replaced
- In between use, store gel pads on reusable tray, gel side down
- Do not rinse gel pads with water
- For additional cooling relief, store gel pads in refrigerator
- Do not use pads with nipple creams or other topical remedies

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