Lansinoh®

DIRECTIONS FOR USE

Step 1

Before feeding or pumping, attach the neck strap to the Silicone Breast Pump and place around your neck to prevent spills during use.

Step 2

Squeeze the center of the Silicone Breast Pump and place over your breast to form an airtight seal. Breastmilk will be collected once your let-down begins.

Step 3

Once your Silicone Breast Pump is full or you have stopped feeding, release by gently squeezing the flange area while securely holding the base.

Step 4

Pour your collected breastmilk into a Lansinoh® Breastmilk Storage Bag or Breastmilk Storage Bottle. Place cap over the top of your Silicone Breast Pump to prevent contamination.

TIPS

Because of the natural way that your body releases breastmilk, the amount of fat in the milk available to baby gradually changes as a feed progresses. We suggest that you mix breastmilk collected with your Lansinoh® Silicone Breast Pump with expressed breastmilk to ensure the maximum nutritional benefits when feeding your little one.

If you are struggling to get your milk flow going, try a Lansinoh® Thera®Pearl® 3-in-1 Breast Therapy to encourage let down and promote faster milk flow.

CLEANING & CARE

Follow these guidelines unless specifically advised otherwise by your healthcare provider. Before first use, place your Silicone Breast Pump in boiling water for 5 minutes, allow to cool, thoroughly rinse and air dry.

- Always wash all components of the product immediately after each use.
- Can be washed in hot soapy water and rinsed in clean water.
- Dishwasher safe on top rack only.
- Safe to clean in a microwave-safe, steam sterilizer bag and special sterilizing machine.

NOTE: DO NOT MICROWAVE HUMAN MILK!

Microwaving alters the composition of the milk. Microwaving also may create "hot spots" in the milk that could burn the infant.

Remove neck strap prior to cleaning. Do not clean with solvents or harsh chemicals. Avoid contact with sharp objects.

WARNING

THIS PRODUCT IS NOT A TOY. The Silicone Breast Pump neck strap is solely designed and intended for wear by adults. Do not place the neck strap around baby's neck. Babies and children should never be left unattended with the Silicone Breast Pump neck strap. Do not let baby chew on clasp. Adult supervision is always required. Discard if damaged.

Lansinoh® is a registered trademark of Lansinoh Laboratories, Inc. © 2022 Lansinoh Laboratories All Rights Reserved. Made in Izmir, Turkey, 35035

Distributed by: Lansinoh Laboratories, Inc. Alexandria, VA 22314 USA, +1 800-292-4794

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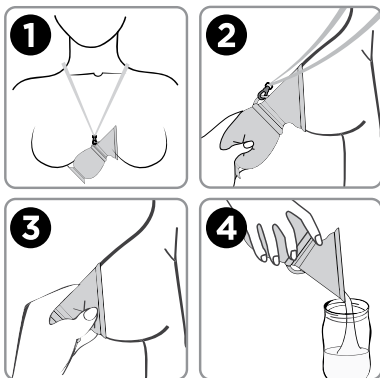
EN INSTRUCTIONS FOR USE

Intended Use of Breast Pump

The Lansinoh® Silicone Breast Pump is intended to collect breastmilk for the purpose of feeding breastmilk to a baby. The Silicone Breast Pump is intended for a single user.

Indications of Lansinoh® Silicone Breast Pump

The Lansinoh® Silicone Breast Pump is designed to catch extra breast milk produced by a mother's opposite breast while breastfeeding. Secure, comfortable suction attaches hands-free to let mom conveniently collect up to 4 ounces of excess breast milk. In addition, the Lansinoh® Silicone Breast Pump has a neck strap and a suction base to prevent spills. The simple design is easy to use, lightweight, dishwasher safe, and easy to clean. When feeding or pumping from one breast, it's common for the other side to leak, and no one likes to see wasted breastmilk! The Lansinoh® Silicone Breast Pump gives you a simple way to collect this precious breastmilk, so you never lose a drop.

When feeding or pumping from one breast, it's common for the other side to leak, and no one likes to see wasted breastmilk! The Lansinoh® Silicone Breast Pump gives you a simple way to collect this precious breastmilk, so you never lose a drop.

Made from super soft 100% food grade BPA free silicone, the Lansinoh® Silicone Breast Pump works by fitting securely to the breast, forming an airtight seal. The seal creates a natural suction, collecting the breastmilk that leaks from the opposite breast while breastfeeding.

This product is not intended to replace a breast pump to maintain supply when away from baby.

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