




REF HCSPUPPYRD

aeromist buddies®

reusable and disposable
nebulizer
compressor kit

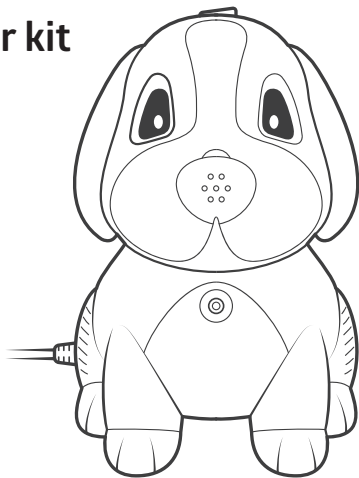

CONTENTS

INTRODUCTION	4
GENERAL PRECAUTIONS	4
ACCESSORIES	5
USER INSTRUCTIONS	6
AIR FILTER CHANGE	7
CLEANING	7
SPECIFICATIONS	8
SPARE PARTS LIST	9
TROUBLESHOOTING GUIDE	10
SYMBOLS KEY	10
5 YEAR LIMITED WARRANTY	11

INTRODUCTION:

Please read the manual carefully before using the device.


- This product is a nebulizer for the inhalation of medical aerosols to treat asthma, bronchitis and other respiratory tract disorders. This nebulizer is suitable for all commonly used solutions. Use only the type and amount of medicine approved by your doctor.
- This product is intended for aerosol therapy only. Any other use is not recommended.

GENERAL PRECAUTIONS:

- This device is not waterproof or water resistant. Do not allow this device to become submerged in water.
- The device should not be touched with wet hands.
- Do not use the device in the presence of flammable anesthetic substances.
- Unplug the device before cleaning.
- Unplug the unit when it is not in use.
- Do not untwist the nebulizer cup during operation.
- Do not leave solution in the nebulizer cup and always empty when it is not in use.
- Do not pour more than 10 mL of solution into nebulizer cup.
- Place the device on a flat surface.
- This product is meant for intermittent use. Do not operate for longer than 30 minutes continuously.
- In order to prevent the risk of cross contamination, the accessories should not be shared by different people.
- Children may only use the device under adult supervision.
- This device can only be repaired by the local dealer's authorized service agents.


NOTE: This device carries a thermal limiter which will shut off the unit to avoid overheating.

ACCESSORIES:

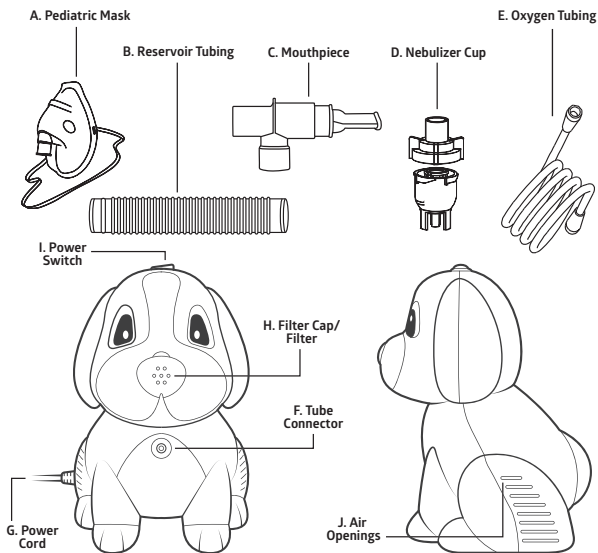

USER INSTRUCTIONS

1. Remove unit from packaging. Open the nebulizer cup (Fig. A) and pour the medication taking into account that the maximum capacity is 10 mL (Fig. B). Twist both pieces and close it (Fig. C).
2. Connect one end of the tubing to the nebulizer cup and the other end to the nebulizer outlet (Fig. D).
3. If using the mouthpiece, connect it to the reservoir tubing.
4. Connect the mask or the mouthpiece to nebulizer cup (Fig. E).
5. Plug in unit and flip ON switch to start treatment.
6. Turn the unit off and unplug it once the treatment has finished.
7. Clean the nebulizer and the accessories as indicated in "Cleaning" section.

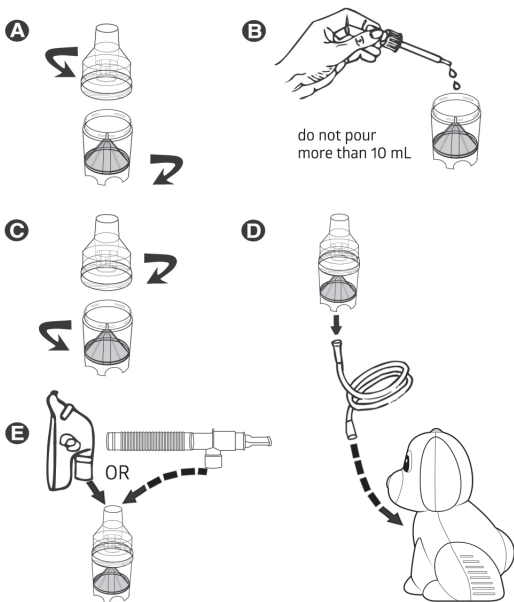

AIR FILTER CHANGE

CAUTION: The nebulizer may get damaged if a dirty or torn filter is used or if it is replaced by another material such as cotton. Do not operate without a filter.

CLEANING

It is recommended that the nebulizer, mouthpiece and mask are thoroughly cleaned with hot water after each use and cleaned with a mild detergent after the last treatment of the day.

If your physician or respiratory therapist specifies a different cleaning procedure, follow their instructions.

Rinsing (after each treatment):

1. Disconnect the oxygen tube, nebulizer cup, mouthpiece and mask from the compressor.
2. Gently twist the nebulizer cup to open it.
3. Rinse the nebulizer cup, mouthpiece and mask with warm tap water.
4. Let air dry on an absorbent piece of paper. Do not dry them with a towel.
5. Reassemble the nebulizer when completely dry and put these parts in a dry, sealed container.

Disinfecting:

Follow these steps to disinfect your nebulizer cup, unless otherwise specified by your physician. It is suggested that the unit is disinfected after the last treatment of the day.

1. Mix 1 part of white vinegar with 3 parts of hot water in a clean container.
2. Completely submerge the nebulizer, mouthpiece, and mask in the solution for 30 minutes.
3. Remove the accessories from the solution and rinse with warm water.
4. Allow to air dry.

Cleaning the Compressor:

WARNING! Unplug the power cord before cleaning. Never submerge the compressor in water or get liquid into the compressor.

Wipe daily with a damp cloth. Do not use any volatile fluid, such as alcohol or benzene, which may damage the finish.

Filter Change:

- Replace only with filters sold as replacements for this unit.
- Do not wash or clean the filter.
- Do not operate the compressor without a filter.
- Change the filter every 30 days or sooner if the filter turns gray.

Changing Procedure:

1. Remove the filter cover.
2. Replace the used filter with a new one, discard the old filter.
3. Replace the filter cover.

SPECIFICATIONS

Power.....	120V/60Hz
Power Consumption.....	180VA
Medication Capacity.....	10 mL
Particle Size	0.5 - 10 μ m
MMAD.....	3 μ m
Sound level	\leq 60dba (1 m away)
Average Nebulization Rate.....	\geq 0.2 mL/min
Compressor Max. Pressure Range.....	35-50 Psi
Operating Pressure Rate.....	10-17 Psi
Liter Flow Range.....	4.0 - 7.5 lpm
Operating Temperature Range.....	10 - 40°C (50 - 104°F)
Operating Humidity Range.....	10-93% RH
Storage.....	Protect from freezing and avoid excessive heat
Dimension (L x W x H).....	196 x 172 x 225 mm
Weight.....	3.2lb (1.45kg)
Standard Accessories.....	Disposable nebulizer cup, reusable nebulizer cup, oxygen tubing, mouthpiece, filters, pediatric mask

The specifications and the outer design are subject to change for improvement reasons without notice. This specification may vary with the kind of medication being used and its viscosity.

SPARE PARTS LIST

Parts	Description
	Disposable and Reusable Nebulizer Cup
	Oxygen Tubing (2)
	Mouthpiece (2)
	Filters (5)
	Pediatric Mask
	Reservoir Tubing (2)
	Carrying Bag

Contact your Medline representative or call 1-800-MEDLINE (800-633-5463) when ordering replacement supplies and parts.

TROUBLESHOOTING GUIDE:

If a problem occurs during operation, please check and correct the following points:

When switching it on, it does not work.	Incorrectly plugged.	Check if the unit is properly connected.
The mist does not come out from the nebulizer cup. Aerosol output is low/reduced.	<ol style="list-style-type: none"> 1. Incorrect amount of medication in nebulizer cup. 2. Tubing disconnected. 3. Nebulizer cup is clogged. 4. The air filter needs to be replaced. 	<ol style="list-style-type: none"> 1. The nebulizer cup's maximum capacity is 10mL. Adjust the volume until the nebulizer functions correctly. 2. Remove excess medication. 3. Connect the tubing. 4. Replace the nebulizer cup. 5. Replace the air filter.

SYMBOLS KEY

On		Class II	
Off	○	Type B	
AC	~	Read the instructions	

5 YEAR LIMITED WARRANTY

Medline Industries, Inc. warrants to the original retail purchaser that this product is free from defects in material and workmanship for a period of 5 years from the date of purchase. This warranty does not cover failures resulting from misuse, abuse, alteration of the Medline Nebulizer System, improper component use or failure to comply with the operating instructions. This warranty does not include the Nebulizer System tubing or accessories. Normal maintenance items and disposable products are not covered under warranty. For non-warranty repairs, the customer will be charged the current repair rate at the time of receipt of unit, and all shipping costs are the responsibility of the purchaser.

Medline Industries, Inc. will either repair or at its option, replace free of charge any parts necessary to correct defects in material or workmanship covered by this limited warranty. Warranty registration cards must be returned within ten (10) days of purchase for this warranty to be effective.

THIS LIMITED WARRANTY IS IN LIEU OF ALL OTHER WARRANTIES, WHETHER ORAL, WRITTEN, EXPRESSED OR IMPLIED. THERE ARE NO WARRANTIES OF MERCHANTABILITY OR FITNESS FOR A PARTICULAR PURPOSE, EXCEPT FOR THOSE IMPLIED WARRANTIES ARISING BY OPERATION OF LAW WHICH ARE LIMITED IN DURATION TO THE 5 YEAR TERM OF THIS WARRANTY. MEDLINE INDUSTRIES, INC.'S LIABILITY, WHETHER BASED ON CONTRACT, TORT, WARRANTY, STRICT LIABILITY, OR ANY OTHER THEORY, SHALL NOT EXCEED THE PRICE OF THE INDIVIDUAL MEDLINE NEBULIZER SYSTEM WHICH IS THE BASIS OF THE CLAIM. IN NO EVENT SHALL MEDLINE INDUSTRIES, INC. BE LIABLE FOR ANY LOSS OF PROFITS, FACILITIES, EQUIPMENT OR OTHER INCIDENTAL, CONSEQUENTIAL OR SPECIAL DAMAGES.

Some states do not allow limitations on how long an implied warranty lasts, or allow the exclusion or limitation of incidental, consequential, or special damages, so the above limitations or exclusions may not apply to you. This warranty gives you specific legal rights. You may also have other rights which vary from state to state.

www.medline.com ©2018 Medline Industries, LP
Medline is a registered trademark of Medline Industries, LP.
Manufactured For: Medline Industries, LP, Three Lakes Drive,
Northfield, IL 60093 Packaged in China.
Products of China and Taiwan. V2 RL21ZBI