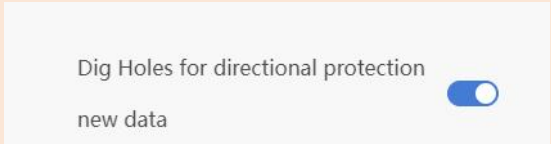


New version software feature			
Software module	State	Function	Function introduction
Settings	Modification	Scan settings	
	Added	Restriction for the angulation of the image capture after digging the hole.	<p>If you turn on the function, the user needs to dig hole in the angulation of the scan body installation. If it is off, the user can dig hole freely.</p> 
		Scanning depth	<p>New version can set the scanning default depth.</p> 
Scanning plan	Added	Implant & Pre preparation scan	<p>Implant and the preparation can be scanned in one case.</p> 
		Implant interference scan body	<p>For installations where the scan bodies are interlaced, this function can be used to scan them separately.</p> 
Acquisition	Added	Rapid heating	<p>Scanning Tip heats up quickly for 30 seconds, then returns to normal heating.</p> 
		Optimize the black tooth	<p>Enhancement of the scanner brightness to improve the scanning of dark/smoking teeth</p> 
		Protect slit between the teeth	<p>It is used to prevent sticking problems by scanning the gap between the teeth. Only applied for this.</p> 
Optimized Part			
a. Music changes after successful matching of occlusion.			