

**numera**<sup>®</sup>  
a **Nice** brand

# Demo Fall Detection



[numera.com](http://numera.com)



# Demo Fall Detection

- New Demo feature allows a user to test Fall Detection on a Libris 2 device.
- Users can have peace of mind knowing that automatic fall detection is enabled on the device and works.
- Steps to perform the demo are simple, easy to follow and safe.
- Provides a 30-minute window for user to test fall detection.
- Demo feature *turns off automatically* after 30 minutes – or can be manually disabled - and device returns to normal Fall mode.

COMPANY CONFIDENTIAL – Silent Call




numera®

# Setting up the Demo Fall Detection - Prerequisites

- Realm *must* be configured to allow for this feature.
  - Dealers can contact Numera Technical Support to request this feature (1.855.546.3399).
- Fall Detection must be enabled on the device.
- Device software must be minimum v2.6.1.
- User must be aware that Demo mode is being enabled for Fall Detection.

# Setting up Demo Fall Detection - Dealer Actions


- To enable Demo Fall Detection (FD)
  - Log into the Numera Dealer Portal
  - Go to the Device page
  - Go to the “Settings” section
  - Hit the “Edit” icon
  - Select “Demo Mode” – On
  - Hit “OK”

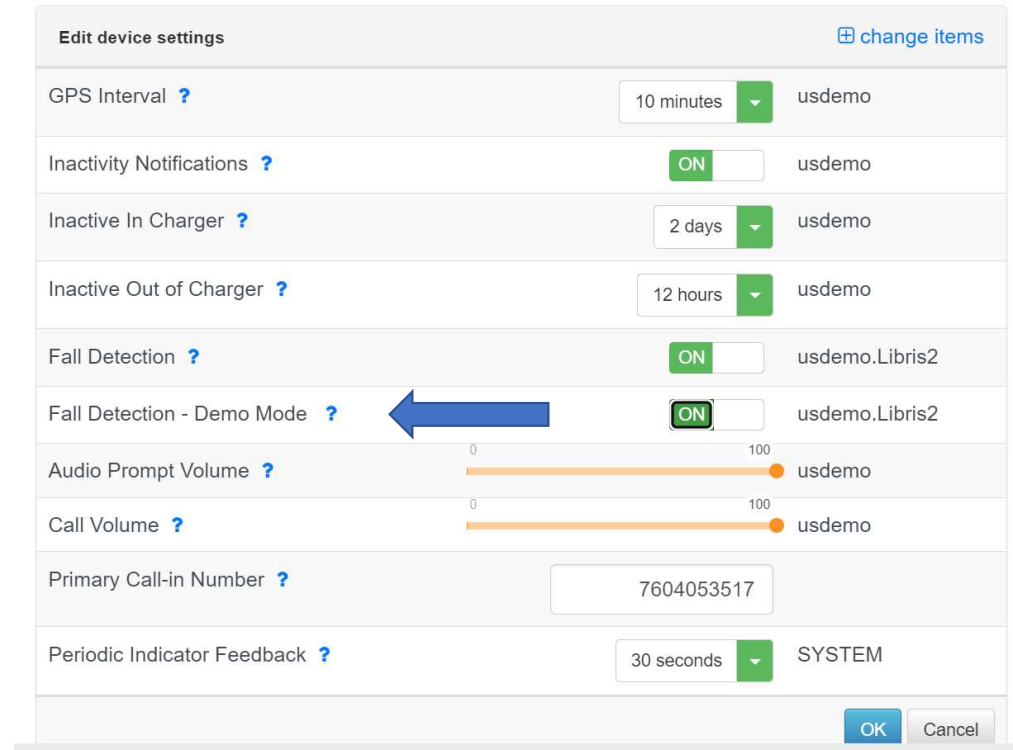

Edit device settings		<a href="#">change items</a>
GPS Interval ?	10 minutes	usdemo
Inactivity Notifications ?	<input checked="" type="checkbox"/> ON	usdemo
Inactive In Charger ?	2 days	usdemo
Inactive Out of Charger ?	12 hours	usdemo
Fall Detection ?	<input checked="" type="checkbox"/> ON	usdemo.Libris2
Fall Detection - Demo Mode ?	<input checked="" type="checkbox"/> ON	usdemo.Libris2
Audio Prompt Volume ?	<input type="range" value="50"/>	usdemo
Call Volume ?	<input type="range" value="50"/>	usdemo
Primary Call-in Number ?	7604053517	
Periodic Indicator Feedback ?	30 seconds	SYSTEM
		<input type="button" value="OK"/> <input type="button" value="Cancel"/>

# Testing Demo Fall Detection – Dealer Actions

- Dealer will confirm they see an event – Demo Fall Detection (FD) Enabled.
  - Click on Locations tab to see the event list
- This action sets the timer for 30 minutes.
- Dealer informs the user that Demo FD is enabled.

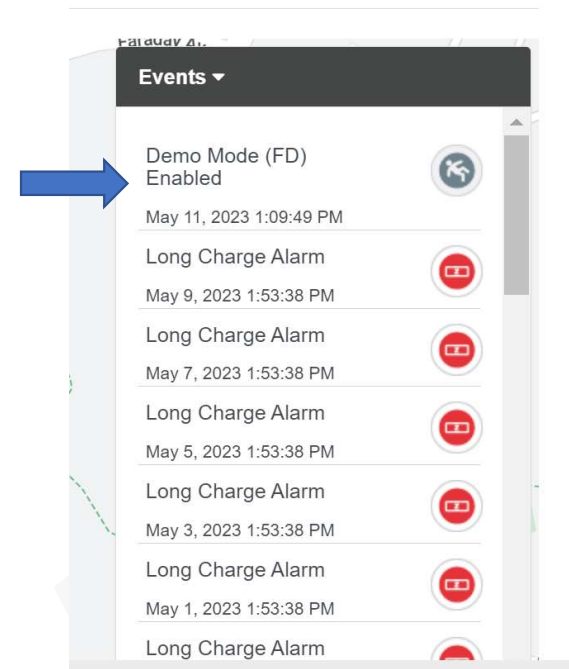

# Testing Demo Fall Detection – User Actions

- User picks up the device.
- From a standing position, user extends their arm out straight, parallel to the ground.
- User drops the device on to a firm sofa or chair (it is okay if it bounces).
- Device must be left on the surface for 3 seconds and can then be picked up.
- Within a minute, the device will enunciate that it detected a fall and place a call to the station.
- User can tell the operators they are testing Fall Detection.

COMPANY CONFIDENTIAL – Silent Call

The logo for Numera, featuring a large, stylized, light gray 'n' in the background. The word 'numera' is written in a blue, lowercase, sans-serif font, with a registered trademark symbol (®) at the end.

numera®

# After Testing is Complete

- User has no further action. They can resume wearing the device.
- System will disable the Demo Fall Detection feature automatically after 30 min – or – dealer can manually disable it.
- Device returns to normal Fall Detection mode.
- Dealer can confirm they see a Demo Fall Detection Disabled event.
- Dealer can reach out to the user telling them the FD demo mode is disabled.

# Fall Detection Demo - FAQs

- What if the demo does not detect a fall?
  - It is possible that it may not detect a fall. If that occurs, you can try again.
- How often can I test demo fall detection?
  - During the 30-minute window, a user can test demo fall detection multiple times.
- Will the 30-minute test window end automatically?
  - The system is designed to end the demo time frame after 30 min. However, dealers can disable the setting sooner, if needed.



# Fall Detection Demo - FAQs

- Will the user get notification that the 30 min window has ended?
  - No. The device does not give any indication, but we recommend the dealer inform the user once the time has ended.
- Will the fall detection demo call a central station?
  - Yes. The fall detection demo will place a call to the central station. In most cases, the operator will have indication that it is a “demo” call. There will also be 2-way voice so the user can communicate that to the operator as well.

# Fall Detection Demo - FAQs

- What if the user has an actual fall during the period the demo mode is enabled? Will the fall be detected?
  - No. The device will not detect actual falls while the demo mode is enabled. However, the user can press the button to call for assistance.

**numera**<sup>®</sup>  
a **Nice** brand

Thank You

[numera.com](http://numera.com)

**numera**<sup>®</sup>